THUASNE SPORT

- fr Bandage rotulien Thuasne Sport
- en Thuasne Sport patellar bandage
- de Patella-Bandage Thuasne Sport
- nl Thuasne Sport knieschijfverband
- it Bendaggio rotuleo Thuasne Sport
- es Venda rotuliana Thuasne Sport
- pt Ligadura rotuliana Thuasne Sport
- da Knæskalsbind Thuasne Sport
- fi Thuasne Sport -polvilumpioside
- sv Thuasne Sport knäskålsförband
- el Επίδεσμος επιγονατίδας Thuasne Sport
- cs Čěšková ortéza Thuasne Sport
- pl Bandaż do owijania rzepki Thuasne Sport
- lv Ceļa skriemeļa apsējs Thuasne Sport
- lt Girnelės juosta "Thuasne Sport"
- et Põlvekedra side Thuasne Sport
- sl Patelarni trak Thuasne Sport
- sk Športová patelárna bandáž Thuasne Sport
- hu Thuasne Sport térdkalács szalag
- bg Пателарен бандаж Thuasne Sport
- ro Bandaj rotulian Thuasne Sport
- ru Коленный ортез Thuasne Sport
- hr Patelarna traka Thuasne Sport
- zh Thuasne Sport 髌骨绷带
- ar Thuasne Sport لفافة رضفية

26 - 50 cm

fr	Circonférence du genou
en	Knee circumference
de	Knieumfang
nl	Omvang van de knie
it	Circonferenza del ginocchio
es	Circunferencia de la rodilla
pt	Circunferência do joelho
da	Omkreds af knæ
fi	Polven ympärys
sv	Omkrets runt knä
el	Περιμέτρος του γόνατος
cs	Obvod kolena
pl	Obwód kolana
lv	Celgala apkārtmērs
lt	Kelio apimtis
et	Põlve ümbermõõt
sl	Obseg kolena
sk	Obvod kolena
hu	Térd körfogata
bg	Обиколка на коляното
ro	Circumferința genunchiului
ru	Диаметр окружности колена
hr	Opseg koljena
zh	膝关节周长
ar	محيط الركبة

fr	Taille unique
en	Universal size
de	Einheitsgröße
nl	Eén maat
it	Taglia unica
es	Talla única
pt	Tamanho único
da	Kun én størrelse
fi	Yksi koko
sv	En storlek
el	Ένα μέγεθος
cs	Univerzální velikost
pl	Rozmiar uniwersalny
lv	Universāls izmērs
lt	Vienas dydis
et	Üks suurus
sl	Enotna velikost
sk	Univerzálna veľkosť
hu	Egy méret
bg	Универсален размер
ro	Mărime universală
ru	Единый размер
hr	Univerzalna veličina
zh	均码
ar	مقاس واحد

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THUASNE SPORT PATELLAR BANDAGE

Description/Destination:

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing of the product.

Patellar compression bandage.

The device is composed of:

- a textile strap combined with a strapping strap providing targeted compression and support,
- a silicone insert positioned over the patellar tendon,
- an adjustable touch fastening system to adjust the strapping strap.

Composition:

Textile components: polyamide - polyurethane - polyethylene - elastan - synthetic rubber (SBR) - polyester.

Rigid components: polyamide - Polyoxymethylene - Silicone.

Properties/Mode of action:

The patellar bandage applies local pressure on the patellar tendon and supports the patella (kneecap), helping to absorb vibrations. Proprioceptive effect thanks to the combination of silicone with dimpling, creating a massage effect, and the fabric strap providing compression.

Indications:

Joint or ligament inflammation of the kneecap, patellar tendinitis (Runner's knee and Jumper's knee).

Mild patellar instability.

Anterior knee or patella pain.

Contraindications:

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions:

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Make sure the knee circumference is compatible with the product circumference.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Consult a healthcare professional in case of uncertain diagnosis.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches..).

In case of doubt before any sports activity, consult a healthcare professional.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect:

This device can cause skin reactions (redness, itching, burns, blisters..) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application:

It is recommended to wear the device directly on the skin, unless contraindicated.

- Unfasten the tightening strap.

- The patellar bandage must be placed over the tendon under the patella (kneecap). The tendon must be located at the centre of the silicone insert.

To adjust the patellar bandage, a removable touch fastening and tightening strap pre-cutting system is available:

- Remove the removable touch fastener ①.
- Cut the tightening strap at the marker corresponding to your size ②.
- Reinstall the removable touch fastener ③.
- Pass the tightening strap back through the buckle then install the patellar bandage.
- Fasten the strap around the knee.
- Make sure the Thuasne logo is on the upper part of the bandage.

Care:

Product can be washed in accordance with the conditions shown on this notice and on the label. Do not machine wash the device more than 30 times. Machine wash at 30°C (delicate programme). Close the self-fastening tabs before washing. If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage:

Store at room temperature, preferably in the original packaging.

Disposal:

Dispose of in accordance with local regulations.

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©Thuasne - 2027901 (2021-03)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186