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Conservet cette notice.

en FIXED / FOLDING / HINGED FOLDING WALKER

Description/Destination
Fixed (W2320) / folding (W2300) / hinged folding (W2310) walker.
This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support.
For indoor use only.

Main components
Frame: aluminum.
Handles: PVC.
Ferrules: synthetic rubber.

Indications
Post-operative and post-traumatic support.
Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.
Degenerative conditions of the lower limbs.

Contraindications
Not suitable for patients weighing > 100 kg.
Do not use in the event of known allergy to any of the components.
Do not use in the event of severe perceptual function deficit or severe balance disorders.

Precautions
 ⚠ Important warnings
 Verify the product's integrity before every use.
 Do not use the device if it is damaged.
 Before using this product, check that:

- it is stable and rigid,
- no components are incorrectly attached or not properly screwed in place.

The health care professional who supplied the product must make sure that it is appropriate for the patient.
It is recommended that a health care professional adjusts the device and supervises the first use.
 Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.
 Do not use the device for any use other than its intended use: risk of injury or damage to equipment.
 Never allow children to play with the product.
 Beware of the risks of pinching when handling the device.
 Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.
 Take care when using the device on slippery surfaces (wet floors or wet tiles, etc.), uneven or sloping surfaces, or unstable surfaces (gravel).
 Do not use on stairs or escalators.
 Do not leave exposed to direct sunlight for a long period of time. Check the temperature of the device surfaces: risk of burns.

Undesirable side-effect

- Risk of discomfort in the event of incorrect adjustment of the device.
- Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use
Unfolding (folding/hinged folding walker).
 Unfold the sides of the frame outwards until the locking mechanism engages (clicks).
 Adjusting the height.
The height of the device must be adjusted with reference to the height of the patient.
 Incorrect adjustment can impair performance and place too much strain on the user.
The top of the handle must be approximately at the height of the wrist fold when the user's arms are alongside the body, wearing usual shoes.

Adjustment:

- Press the push-button while pulling the tube in order to adjust the height.
- Check that the push-button springs back out of the tube.
- Make sure that all four feet are adjusted to the same height.
- Then make sure that the walker is locked in position by pressing it with increasing force on the ground.

Folding (folding/hinged folding walker):
 To fold the walker again, press plastic piece A in the middle of the top crossbar and fold the sides of the frame inwards:

Use:
 To walk with the walker, start by standing up facing it and lean on the two handles.
 Fixed/folding walker: lift the walker and place it a little further in front of you, then move forward, holding the device firmly with both hands. Repeat the action.
 Hinged folding walker: move one side forward with one of the two handles, repeat the action on the other side, then move forward, holding the device firmly with both hands.
Two step lengths are possible:

- long steps (21 cm); default setting when the walker is opened,
- shorter steps (12 cm); once the walker is open, press plastic piece A and pull the frame a little wider until it clicks.

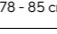
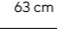
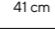
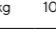
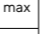
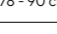
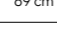
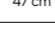
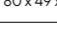
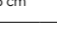
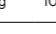
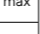
⚠ Important warnings
 Do not attach handbags, carrier bags or other objects to your device.
 The device must not be used as an aid to get up from the sitting position: risk of unbalancing.

Maintenance and care
Maintenance
 If any part of the device becomes detached, loose or visibly damaged, contact a specialized dealer or health care professional.
 If necessary, replacement parts and original accessories are available from your Thuasne product dealer.

Care
 Clean with a mild cleaning agent and a soft cloth.
 Do not clean the handles or the ferrules with greasy substances: risk of slipping.
 If necessary, use a mild contact disinfectant or 70° alcohol.
In an institution, follow the recommendations of the hygiene manager.

Storage
 Store at room temperature, in a dry, ventilated place.

Disposal
 Dispose of in accordance with local regulations.

Technical characteristics:	Adjustment height	Maximum width	Maximum depth	Folded walker dimensions	Total weight	Maximum user weight
						
Fixed walker W2320	78 - 85 cm	63 cm	41 cm	-	1,8 kg	100 kg max
						
Folding walker W2300	78 - 90 cm	69 cm	47 cm	80 x 49 x 18.5 cm	3 kg	100 kg max
						
Hinged folding walker W2310	78 - 90 cm	69 cm	47 cm	79.5 x 47.5 x 19.5 cm	3 kg	100 kg max

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