THUASNE

Elastocalx

D: Viscoelastische Fersenschalen

GB/USA: Visco-elastic heel cushions

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I: Tallonette in viscoelastico

E: Taloneras viscoelásticas

P: Calcanheiras viscoelásticas

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USA

Purpose:

The Elastocalx heel cushions are manufactured using highly elastic silicone and provide optimum shock absorption for ankle and knee joints, hip and spinal column. Elastocalx alleviates stress on the joints, ligaments, muscles and tendons. The heel cushions may only be used for foot treatment. The different models each have specific modes of action.

Indications/Specific modes of action:

Elastocalx N (47980)

Osteoarthritis of the ankle, knee and hip joints, degenerative spinal diseases.

Relief after ankle, knee and hip joint operations, irritations of the Achilles tendon, myotendopathies (long-term, recurrent pain), strain syndromes, differences in leg length, scoliotic pelvis.

Elastocalx S/P (47981)

For the correction of supination (raised inside margin of the foot) or pronation (drop in the inside margin of the foot) in the foot and in cases of valgus or varus deformity of the heel (knock-kneed or bow-legged).

Elastocalx Plus (47982)

Relief from heel spur (bony spur formation under the heel) pressure by means of a particularly soft silicone insert in the area of the heel spur.

Elastocalx Flex (47991)

Relief from heel spur (bony spur formation under the heel) pressure by means of an open, honeycomb construction covering a large area in the region of the heel spur.

Contraindications/Side effects:

Larger skin lesions or diseases in the heel area.

No side effects are as yet known when applied and worn as intended.

Precautions/other advice:

Do not bring into contact with open wounds. Follow the advice of the professional who prescribed or supplied the product.

Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Insertion of the heel cushions:

General and Elastocalx N

The heel cushion and size required for your symptoms was selected by your doctor/hospital. Place the heel cushion flat inside the shoe and press the back margin of the cushion against the heel of the shoe.

Elastocalx Plus

The heel cushions are marked on the bottom with R and L for the right and left shoes.

Elastocalx Flex

Fit left or right.

Elastocalx S/P

In cases of pronation (e. g. valgus) of the foot, place the thick side of the cushion under the inside of the heel, in cases of supination, place the thick side of the cushion under the outside of the heel.

For bow legs (varus, e.g. medial arthritis in the knee joint), place the thick side of the cushion under the outside of the heel (pronate), for knock knees, place the thick side of the cushion under the inside of the heel (supinate).

Please use the second heel cushion in the same way.

Care:

Wash off regularly with lukewarm soapy water (mild detergent). Do not use either aggressive detergents or bleach. Wipe dry with a suitable cloth.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH