THUASNE

# GenuAction

<b>fr</b>	Genouillère élastique proprioceptive.....	7
<b>en</b>	Proprioceptive elastic knee support .....	9
<b>de</b>	Elastische propriozeptive Kniebandage.....	10
<b>nl</b>	Proprioceptieve elastische kniebrace .....	13
<b>it</b>	Ginocchiera elastica propriocectiva.....	15
<b>es</b>	Rodillera elástica propioceptiva.....	16
<b>pt</b>	Joelheira elástica proprioceptiva .....	18
<b>da</b>	Proprioceptisk, elastisk knæbind .....	20
<b>fi</b>	Proprioseptiivinen joustava polvituki.....	22
<b>sv</b>	Proprioceptivt elastisk knästöd.....	24
<b>el</b>	Ελαστική ιδιοδεκτική επιγονατίδα .....	26
<b>cs</b>	Elastická proprioceptivní kolenní ortéza.....	28
<b>pl</b>	Elastyczna proprioceptywna orteza stawu kolanowego .....	30
<b>lv</b>	Proprioceptīva elastīga ceļgala ortoze .....	32
<b>lt</b>	Tamprus propriocepriją gerinantis kelio įtvaras .....	34
<b>et</b>	Propriotseptiivne elastne põlvetugi .....	36
<b>sl</b>	Elastična proprioceptivna opornica.....	38
<b>sk</b>	Elastická proprioceptívna kolenná ortéza .....	40
<b>hu</b>	Proprioceptív rugalmas térdrögzítő .....	42
<b>bg</b>	Проприоцентивна еластична скоба за коляното.....	44
<b>ro</b>	Genunchieră elastică proprioceptivă.....	46
<b>ru</b>	Эластичный проприоцептивный наколенник.....	48
<b>hr</b>	Elastični proprioceptivni steznik za koljeno.....	50
<b>zh</b>	弹力本体感护膝.....	52
<b>ar</b>	دعامة ركبة مرنة متعلقة باستقبال الحس العميق.....	55

	0	28 - 31 cm
	1	32 - 34 cm
	2	35 - 37 cm
	3	38 - 41 cm
	4	42 - 45 cm
	5	46 - 49 cm
	6	50 - 53 cm

<b>fr</b>	Circonférence de genou	Maintien
<b>en</b>	Knee circumference	Support
<b>de</b>	Knieumfang	Halt
<b>nl</b>	Omvang van de knie	Fixatie
<b>it</b>	Circonferenza del ginocchio	Sostegno
<b>es</b>	Circunferencia de la rodilla	Sujeción
<b>pt</b>	Circunferência do joelho	Manutenção
<b>da</b>	Omkreds af knæ	Støtte
<b>fi</b>	Polven ympärys	Tuki
<b>sv</b>	Omkrets runt knäet	Stöd
<b>el</b>	Περίμετρος του γόνατος	Συγκράτηση
<b>cs</b>	Obvod kolena	Stabilizace
<b>pl</b>	Obwód kolana	Podtrzymywanie
<b>lv</b>	Celgala apkārtmērs	Atbalsts
<b>lt</b>	Kelio apimtis	Palaikymas
<b>et</b>	Põlve ümbermõõt	Hooldus
<b>sl</b>	Obseg kolena	Oprijem
<b>sk</b>	Obvod kolena	Podpora
<b>hu</b>	Térd körfogata	Támasz
<b>bg</b>	Обиколка на коляното	Поддръжка
<b>ro</b>	Circumferința genunchiului	Suținere
<b>ru</b>	Диаметр окружности колена	Стабилизация
<b>hr</b>	Opseg koljena	Potpورا
<b>zh</b>	膝关节周长	稳定
<b>ar</b>	محيط الركبة	الدعم

Compression	Effet proprioceptif
Compression	Proprioceptive effect
Kompression	Propriozeptive Wirkung
Compressie	Proprioceptieve werking
Compressione	Effetto proprioceptivo
Compresión	Efecto propioceptivo
Compressão	Efeito proprioceptivo
Kompression	Proprioceptisk effekt
Kompressio	Liikeaistiin kohdistuva vaikutus
Kompression	Proprioceptiv effekt
Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
Komprese	Proprioceptivní účinek
Kompresja	Propriocepccja
Kompresija	Proprioceptīvs efekts
Kompresija	Propriocepçinis poveikis
Kompressioon	Proprioceptiivne toime
Kompresija	Proprioceptivni učinek
Kompresia	Proprioceptívny účinok
Szorítás	Proprioceptív hatás
Компрессия	Проприоцептивен ефект
Compressie	Efect proprioceptiv
Компрессия	Проприоцептивный эффект
Kompresija	Učinak proprioceptivnog vježbanja
压紧	本体感

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---

en

## **PROPRIOCEPTIVE ELASTIC KNEE SUPPORT**

### **Description/Destination**

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 7 sizes.

The device is composed of an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.

### **Composition**

Polyester - polyurethane - cotton - polyamide - elastane.

### **Properties/Mode of action**

Elastic knit (compression: class 3 - 20-36 mmHg / 27.1-48 hPa) ensuring good support and proprioception.

### **Indications**

Knee pain and/or swelling management.

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Conservative treatment of mild knee ligament injuries.

Compression for scar/burns healing.

### **Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

#### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Instructions for use/Application**

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace on the leg with your foot extended, making sure that the patella is properly positioned in the flexible knitted zone provided for this purpose.

#### **Care**

Machine washable at 30°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2024201 (2021-01)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186