

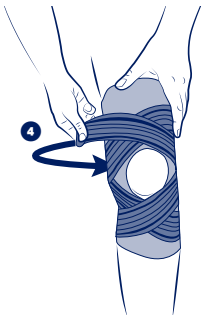
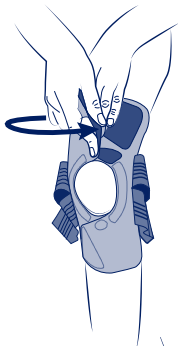
THUASNE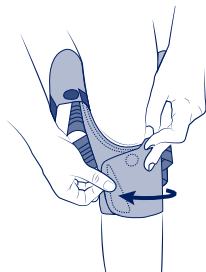

SPORT

fr	Genouillère strapping ouverte Thuasne Sport	4
en	Thuasne Sport open strapping knee brace.....	5
de	Offenene Thuasne Sport-Strapping-Kniebandage.....	7
nl	Thuasne Sport open kniebrace met elastische banden ..	9
it	Ginocchiera strapping aperta Thuasne Sport.....	11
es	Rodillera strapping abierta Thuasne Sport	13
pt	Joelheira de strapping aberta Thuasne Sport.....	15
da	Åbent knæbind med rem Thuasne Sport	16
fi	Nauhallinen avoin Thuasne Sport -polvituki.....	18
sv	Knästöd med öppet elastiskt band Thuasne Sport....	20
el	Επιγονατίδα strapping ανοικτή Thuasne Sport	22
cs	Otevřená kolenní ortéza strapovací Thuasne Sport ..	24
pl	Bandażowa orteza kolanowa otwarta Thuasne Sport... 	25
lv	Atvērtā elastīga ceļa ortoze Thuasne Sport	27
lt	Apjuosiamas atviras kelio įtvaras „Thuasne Sport“	29
et	Fikseerivate rihmadega avatud põlvepealsega põlvekaitse Thuasne Sport	30
sl	Odperta opornica iz trakov za koleno Thuasne Sport..	32
sk	Otvorená kolenná ortéza s popruhmi Thuasne Sport	34
hu	Thuasne Sport nyitott pántos térdmerezítő.....	35
bg	Отворена пристягаща наколенка Thuasne Sport... 	37
ro	Genunchieră deschisă cu benzi ajustabile Thuasne Sport	39
ru	Открытый ременный наколенник Thuasne Sport... 	41
hr	Otvoreni steznik za koljeno s trakama Thuasne Sport	43
zh	Thuasne Sport 开放式肌内效护膝.....	45
ar	Thuasne Sport داعم ركبة مفتوح مزود بشرائط الشد	47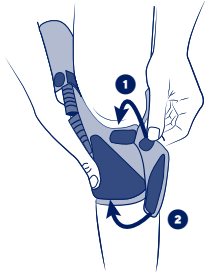

 cm	S 32 - 36	M 37 - 40	L 41 - 44	XL 45 - 48
---	---------------------	---------------------	---------------------	----------------------

fr	Circonférence de genou en légère flexion
en	Knee circumference in slight flexion
de	Kniefumfang bei leichter Beugung
nl	Omvang van de licht geplooidie knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Celgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenktu kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
ru	Обхват колена в состоянии небольшого сгибания
hr	Obuhvaća koljeno u blagoj fleksiji
zh	轻度弯曲的膝关节周长
ar	محيط الركبة في حالة النسي الخفيف

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THUASNE SPORT OPEN STRAPPING KNEE BRACE

Description/purpose

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 4 sizes.

The device is composed of:

- flexible lateral reinforcement,
- a double functional strapping ensuring good lateral support,
- un tissu en synthetic rubber (SBR).

Composition

Textile components: polyamide - synthetic rubber (SBR) - polyester - thermoplastic polyurethane - elastan - ethylene vinyl acetate - polyethylene - Natural latex.

Rigid components: polyamide - Zinc alloy.

Properties/Mode of action

Stabilisation of the knee ligaments provided by the double strapping system.

Anti-slip system to prevent the knee brace slipping.

Opening to make fitting easier.

Limitation of the heat loss by the synthetic rubber (SBR).

Indications

Joint instability/laxity.

Knee pain and/or swelling management.

Sport activities resumption.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Contains natural latex.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Do not wear the product in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

To avoid damage to the knit fabric, please make sure that the self-gripping pads are always fastened to the designated self-gripping surfaces when putting it on and taking it off.

Close the tissue fabric's sheath onto the joint as slightly bent, starting from the low portion.

Adjust the kneecap adjustment first ❶ and then the fastening tab ❷.

Repeat the operations for the upper portion.

Once in place, the knee brace clamps and holds the kneecap.

Tighten the lower cross-over straps and wrap them around the rear of the knee to fasten them ❸.

Then tighten the upper straps and fasten them at rear of the knee support ❹.

Make sure the straps do not go on the kneecap. Release it if necessary. For comfort during wear, make sure the straps are not creased.

If you feel the need to tighten the straps during an activity, make sure you tighten them following the steps indicated in the fitting instructions.

Care

Close the self-fastening tabs before washing. Machine washable at 30 °C. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2034901 (2022-08)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186