
THUASNE










Lombamum[®]

fr	Ceinture lombo-sacrale de maternité	5
en	Lumbo-sacral belt for maternity	5
de	Lumbosakral-Stützgürtel für Schwangere	6
nl	Lumbosacrale zwangerschapsbrace	7
it	Cintura lombo-sacrale per gravidanza	8
es	Faja lumbosacral de maternidad	9
pt	Cinta lombossacral para grávidas	10
da	Lumbosakralt graviditetsbælte	11
fi	Lannerangan ja ristiselän tukivyö raskausajalle	12
sv	Lumbosakralt stödbälte graviditet	13
el	Ζώνη οσφύος μητρότητας	13
cs	Těhotenský bederní pás	14
pl	Orteza lędźwiowo-krzyżowa ciążowa	15
lv	Grūtniecības jostasvietas un krustu daļas josta	16
lt	Juosmens ir kryžkaulio diržas nėščiosioms	17
et	Rasedusaegne nimme-ristлуу vöö	18
sl	Nosečniški oporni križno-ledveni pas	19
sk	Tehotenský lombo-sakrálny pás	20
hu	Terhességig ágyék-keresztcsonti fűző	20
bg	Лумбо-сакрален колан за бременни	21
ro	Centură lumbosacrală pentru gravide	22
ru	Пояснично-крестцовый корсет для беременных	23
hr	Lumbalno-sakralni potporni pojas tijekom trudnoće	24
zh	产妇腰部支撑带	25
ar	حزام للصدر وأسفل الظهر للولادة	26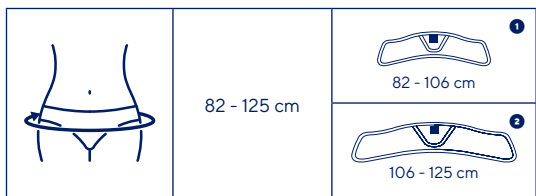

fr	Tour de hanches	Hauteur : 21 cm
en	Hip circumference	Height: 21 cm
de	Hüftumfang	Höhe: 21 cm
nl	Heupomtrek	Hoogte: 21 cm
it	Circonferenza fianchi	Altezza: 21 cm
es	Circunferencia de cadera	Altura: 21 cm
pt	Perímetro de anca	Altura: 21 cm
da	Høftemål	Højde: 21 cm
fi	Lantionympärys	Korkeus: 21 cm
sv	Höftvidd	Höjd: 21 cm
el	Περιφέρεια γοφών	Ύψος: 21 εκ.
cs	Obvod boků	Výška: 21 cm
pl	Obwód w biodrach	Wysokość: 21 cm
lv	Gurnu apkārtmērs	Augstums: 21 cm
lt	Klubų apimtis	Aukštis: 21 cm
et	Puusaübermõõt	Kõrgus: 21 cm
sl	Obseg bokov	Višina: 21 cm
sk	Obvod panvy	Výška: 21 cm
hu	Csípőbőség	Magasság: 21 cm
bg	Обиколка на ханша	Височина: 21 cm
ro	Circumferința șoldurilor	Înălțime: 21 cm
ru	Окружность бедер	Высота: 21 cm
hr	Obujam bokova	Visina: 21 cm
zh	臀围	高度 : 21 cm
ar	محيط الوركين	الارتفاع: 21 سم

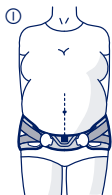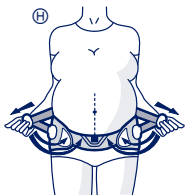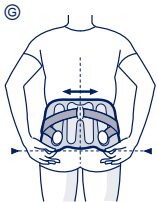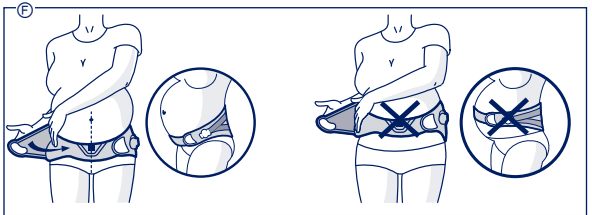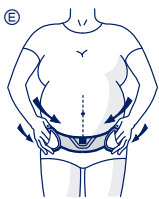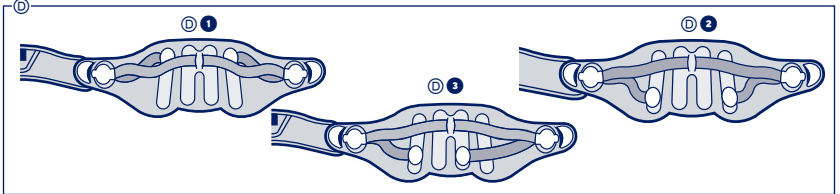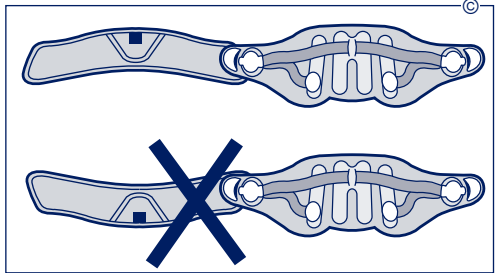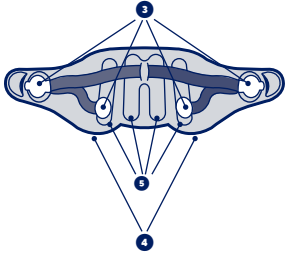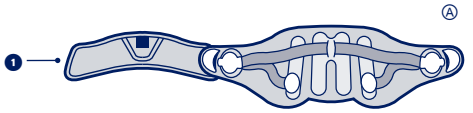fr	Soutien	Décharge	Effet proprioceptif
en	Support	Off-loading	Proprioceptive effect
de	Stütze	Entlastung	Propriozeptive Wirkung
nl	Ondersteuning	Losmaken	Proprioceptieve werking
it	Sostegno	Scarico	Effetto proprioceptivo
es	Sujeción	Descarga	Efecto propioceptivo
pt	Suporte	Alívio do peso	Efeito proprioceptivo
da	Støtte	Aflastning	Proprioceptisk effekt
fi	Tuki	Vapautus	Liikeaistiin kohdistuva vaikutus
sv	Stöd	Avlastning	Proprioceptiv effekt
el	Υποστήριξη	Αποφόρτιση	Αποτέλεσμα ιδιοακουστικότητας
cs	Podpora	Snižení zátěže	Proprioceptivní účinek
pl	Podtrzymywanie	Odciążenie	Propriocepcja
lv	Atbalsts	Atbrīvošana	Proprioceptīvs efekts
lt	Palaikymas	Apkrovos sumažinimas	Propriocepčinis poveikis
et	Toetus	Koormuse vähendamine	Proprioceptiivne toime
sl	Podpora	Razbremenitev	Proprioceptivni učinek
sk	Podpora	Uvoľnenie	Proprioceptívny účinok
hu	Megtámasztás	Tehermentesítés	Proprioceptív hatás
bg	Поддръжка	Отбременяване	Проприоцептивен ефект
ro	Susținere	Descărcare	Efect proprioceptiv
ru	Поддержка	Разгрузка	Проприоцептивный эффект
hr	Potpورا	Rasterećenje	Učinak proprioceptivnog vježbanja
zh	支撐	减压	本体感
ar	الدعم	تخفيف نقل	مفعول استقبال الحس العميق

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LUMBO-SACRAL BELT FOR MATERNITY

Description/Intended users

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Comfort bamboo fibre lining.

Two abdominal support bands of different sizes (one band corresponding to a hip measurement of < 106 cm ¹, the second corresponding to a hip measurement of > 106 cm). ²

Adjustable, doubled additional strap with discs. ③

Centering tabs. ④

Comfortable dorsal stays.

Textile components: polyamide - polyurethane - polyester - elastane - elastodiene - viscose.

Non-textile components: brushed tempered steel - polyoxymethylene.

Properties/Mode of action

The device relieves the back from the baby's weight.

The device ensures an adapted abdominal support: the abdominal support bands closely fit the body throughout the pregnancy and provide firm support for the abdomen.

Indications

Pregnancy-related low back pain.

Pregnancy-related pelvic girdle pain (posterior pelvic pain, sacro-iliac pain, symphyseal pain).

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Wear the device above thin clothing.

Do not use the device if it is damaged.

Strictly comply with your healthcare professional's prescription and recommendations for use.

It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

It is recommended to adequately tighten the device to achieve support without excessive compression.

Undesirable side-effects

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Preparing the device

The abdominal support band already in place on the belt is the band corresponding to a hip measurement of < 106 cm. ①

If it is the right size for you, go straight to the fitting instructions. If your hip measurement is greater than 106 cm: remove the abdominal band in place on the belt.

Replace it with the other one then attach it to the dorsal part of the belt using one of the two self-fastening parts. ②

The two labels must face the same way.

The point of the "V" on the middle of the band must be pointing downwards. ③

Adjustment of the additional strap ④

It is also possible to change the position of the discs located at the back:

- If they are positioned high up, it is the top of your back that will be supported ⑤ ①

- If they are positioned low down, it is the bottom of your back that will be supported ⑥ ②

In the case of sciatica, it is possible to provide more active support for the painful side by adjusting the additional strap located on the same side. ⑦ ③

Fitting the device

Position the belt using the finger loop. ⑧

The point of the "V" must be pointing downwards.

Fasten the belt and adjust it: the "V" must be vertically in line with the navel. ⑨

The belt must be centred on the spine: the dorsal stays should be positioned on either side of the spine. ⑩

If it is not, adjust using the centering tabs located on the back. The bottom of the belt must stop at the top of the crease between the buttocks.

For an adapted support, you can use the additional strap.

Pull the additional strap simultaneously and evenly. ⑪

Attach the tabs on the centre of the abdominal band. ⑫

The level of support can be adjusted as required throughout the day.

Care

Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2023202 (2022-06)
THUASNE UK Ltd