THUASNE

Ligacast® Ice*

fr	Atelle stabilisatrice de cheville avec poches de froid.....	5
en	Stabilising ankle brace with cold packs.....	7
de	Orthese zur Stabilisierung des Sprunggelenks mit Kältepacks.....	8
nl	Stabilisatiebrace voor de enkel met ijskompresen.....	11
it	Tutore stabilizzatore per caviglia con inserti in gel.....	13
es	Férula estabilizadora de tobillo con bolsas de frío.....	15
pt	Tala estabilizadora do tornozelo com bolsas de frio.....	17
da	Stabiliserende ankelskinne med køleposer.....	19
fi	Stabiloiva nilkkatuki kylmätaskuilla.....	20
sv	Stabiliseringsskena för ankeln med kylpåsar.....	22
el	Νάρθηκας σταθεροποίησης ποδοκνημικής με θήκες ψύχους.....	24
cs	Stabilizační ortéza kotníku s chladicími kapsami.....	26
pl	Orteza do stabilizacji kostki z wkładkami chłodzącymi.....	28
lv	Potītes stabilizēšanas saite ar aukstumpakām.....	30
lt	Stabilizuojantis kulkėnies įtvaras su šaldomaisiais paketais.....	32
et	Hüppeliigese lahas külmakottidega.....	34
sl	Opornica za stabilizacijo gležnja s hladilnimi blazinicami.....	36
sk	Stabilizačná členková dlaha s chladivými obkladmi.....	38
hu	Stabilizáló bokarögzítő hűtőpárnákkal.....	40
bg	Стабилизираща шина за глезен със студени компреси.....	42
ro	Atelă stabilizatoare pentru gleznă cu pungi cu gel pentru terapie rece.....	44
ru	Стабилизирующий ортез для лодыжки с охлаждающими пакетами.....	46
hr	Udlaga za stabilizaciju gležnja s hladnim gel oblogama.....	49
zh	帶冷袋的腳踝穩定夾板.....	51
ar	جبيرة تثبيت الكاحل مزودة بأكياس باردة.....	55

	0	< 140 cm
	1	> 140 cm

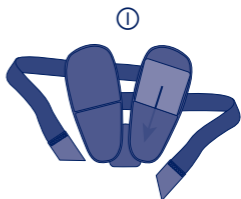
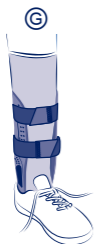
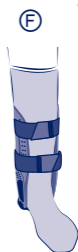
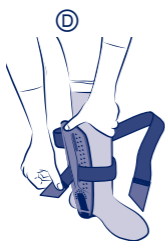
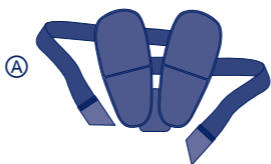
fr	Taille du patient
en	Patient height
de	Größe des Patienten
nl	Maat van de patiënt
it	Altezza del paziente
es	Altura del paciente
pt	Tamanho do paciente
da	Patientens højde
fi	Potilaan koko
sv	Patientens storlek
el	Μέγεθος ασθενούς
cs	Výška pacienta
pl	Rozmiar pacjenta
lv	Pacienta augums
lt	Paciento dydis
et	Patsiendi suurus
sl	Velikost pacienta
sk	Veľkosť pacienta
hu	A beteg mérete
bg	Размер на пациента
ro	Talia pacientului
ru	Размер пациента
hr	Veličina pacijenta
zh	患者身高
ar	مقاس المريض

fr	Stabilisation
en	Stabilisation
de	Stabilisierung
nl	Stabilisatie
it	Stabilizzazione
es	Estabilización
pt	Estabilização
da	Stabilisering
fi	Stabiloi
sv	Stabilisering
el	Σταθεροποίηση
cs	Stabilizace
pl	Stabilizacja
lv	Stabilizēšana
lt	Stabilizavimas
et	Stabiliseerimine
sl	Stabiliziranje
sk	Stabilizácia
hu	Stabilizálás
bg	Стабилизация
ro	Stabilizare
ru	Стабилизация
hr	Stabilizacija
zh	稳定
ar	التثبيت

Cryothérapie
Cryotherapy
Kältetherapie
Cryotherapie
Crioterapia
Crioterapia
Crioterapia
Kryoterapi
Kryoterapia
Kryoterapi
Κρυοθεραπεία
Kryoterapie
Krioterapia
Krioterapija
Šalčio terapija
Krūoterapia
Hladna terapija
Kryoterapia
Krioterápia
Криотерапия
Crioterapie
Криотерапия
Krioterapija
冷疗
معالجة بالبرد

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Conserver cette notice.

en

STABILISING ANKLE BRACE WITH COLD PACKS

Description/Destination

This device is an ankle brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Rigid components: polypropylene - polyurethane foam.

Textile components: polyester - polyamide.

Cold packs: polyvinyl chloride - water - glycerol - carboxymethyl cellulose.

Properties/Mode of action

Lateral rigid plastic shells help stabilise the joint.

Precise fit thanks to adjustable straps.

Heel loop with width adjustment.

Removable foam pads with extra protective layer on the malleoli.

Removable cold packs for cryotherapy treatment.

Indications

Moderate to severe ankle sprain.

Chronic ankle instability.

Ankle pain/oedema.

Cold therapy following ankle ligament injuries.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in case of severe neuropathy conditions.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product in a medical imaging machine.

Do not wear the product when driving a vehicle.

It is recommended to adequately tighten the device to achieve support/stabilisation without restricting blood circulation.

Cold packs for external use only, do not ingest contents.

Do not use the cold pack if it is damaged or pierced.

Use under adult supervision, keep out of children's reach.

Do not use on fragile, insensitive skin or open wounds.

Do not apply the cold pack directly on the skin.

Do not place excessive weight on the cold pack.

Do not sit or lying down on the cold pack.

If the cold pack gel comes into contact with the eyes or skin, rinse immediately with plenty of water and consult a healthcare professional.

Do not heat the cold packs.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Possible risk of ulcers.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

The fitting of the device must be done while seated.

The systematic use of a long sock is recommended when wearing the device. The use of a closed shoe is recommended to strengthen the joint stabilisation.

Loosen the straps. Ⓐ

Place the ankle between the two lateral shells ensuring the shells are properly centered. Ⓑ

If necessary, adjust the width of the heel strap, using the self-fastening areas located on the lower part of the lateral shells. Ⓒ

Tighten the lower strap and fasten securely using the self-fastening closure. Ⓓ

Tighten the upper strap and fasten securely using the self-fastening closure. Ⓔ

If necessary, readjust the straps. Ⓕ

Loosen the shoe completely.

Carefully slide the foot with the device into the shoe.

Tighten the shoe. Ⓖ

If necessary, readjust the straps.

Use of cold packs for cryotherapy:

The brace can be used without the cold packs.

It is possible to use only one cold pack depending on the location of the oedema.

Place the cold packs flat in a freezer compartment (-20°C) for 30 minutes to 2 hours before use.

To avoid the risk of ice burns/frostbite, leave the cold packs for a few minutes at room temperature before using them.

For more comfort, massage the cold pack to make it more flexible before positioning it in the brace. Ⓖ

Slip the cold packs into the pocket at the bottom of the foam pads. Ⓡ

Always check cold packs temperature before application.

To avoid the risk of ice burns/frostbite, do not apply the cold packs for more than 20 minutes and wait at least 20 minutes before applying again.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label.

Shells and cold packs: wipe with a damp cloth.

Foam padding: remove the foam pads from the shells before washing.

Remove the cold packs from the inserts before washing.

Replace in the original location before next use. Machine washable at 30°C (delicate programme). The device has been tested over 20 wash cycles. If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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V prípade potreby upravte šírku pásu pod pätou pomocou suchých zipsov umiestnených v spodnej časti bočných dláh. ©